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Instructions for use Embedding Cassettes

apply to the products embedding cassettes unsterile in all variants and colours, as well as pre-stacked on a tape or in a magazine for use on printers

Product description/Intended use

The embedding cassettes in the versions Universal, Biopsy, Bionet and Standard as well as the mentioned versions as laser embedding cassette are used to hold pathological/histological specimens from the human body. The products are specimen containers into which the tissue samples can be placed, prepared for laboratory analysis and further processed in the laboratory. In addition, the embedding cassettes can be used in the further course of sample processing as block holders for the paraffin wax block in which the sample was previously embedded for cutting on the microtome. Important information about the patient's health can be obtained from the pathological/histological sample.

All products from the product group embedding cassettes are intended exclusively for single use in a professional environment by medically qualified personnel.

Performance features

The embedding cassette is a CE-marked IVD product, which is grouped in class A. The embedding cassettes are made of polyoxymethylene and available in different colours. Furthermore, the cassettes are pre-stacked on a tape or in a magazine available to be used on suitable printers.

In the case of tissues containing hard substances (bones, teeth), it is not advisable to treat the sample stored in the embedding cassette with decalcifying solution containing formic acid or trichloroacetic acid.

Notes

- Wear protective gloves and, if necessary, other protective clothing!
- Do not use damaged or already used embedding cassettes.
- Any violent handling of the product must be avoided.
- Use embedding cassettes only within the specified shelf life.
- The further storage or disposal of the used or contaminated embedding cassettes is carried out according to the regulations of the facility in the suitable disposal container.
- The product is for single use only!

Application

The labelling options using a printer can be found in the specifications of the respective printer manufacturer. When marking by hand, we recommend using a pencil with the HB hardness grade.

Prepare the opened embedding cassette for sample collection and ensure that the filter paper is correctly inserted. Place the removed sample in the cassette and close the lid. Further sample processing steps are not subject of these instructions for use.

Storage

Store dry at +5°C to 25°C and protect from direct sunlight and humidity.

All serious incidents related to the product shall be reported to the manufacturer and the responsible authority of the member state where the user and/or patient is registered.

Symbol meaning

	Item number		Protect from sunlight
	Lot in in YYX0000		Protect from rain
	Shelf life in YYYY-MM		Follow information documents
	Manufacturer		Storage temperature
	For single use only		IVD product
	CE-mark		

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KABE LABORTECHNIK GmbH