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Instructions for use Sterile embedding cassettes

valid for the products embedding cassettes single/to 2 pcs. sterile packed in all variants and colours

Product description/Intended use

The embedding cassettes in the versions Universal, Biopsy, Bionet and Standard are used to hold pathological/histological specimens from the human body. The products are sterile specimen containers into which the tissue samples can be placed, prepared for laboratory analysis and further processed in the laboratory. In addition, the embedding cassettes can be used in the further course of sample processing as block holders for the paraffin wax block in which the sample was previously embedded for cutting on the microtome.

Depending on the design (e.g. biopsy), a double-layer filter paper is already inserted in the embedding cassette for safe storage of small histological specimens.

The embedding cassettes are sterilized by gassing with ethylene oxide and are thus suitable for use in sensitive areas, e.g. ORs.

The embedding cassettes are only products for single use.

The use of the embedding cassettes is intended only in a professional environment by medical professionals.

Performance features

The embedding cassette, individually or sterilely packed in 2 pcs, is made of polyoxymethylene, the inserted filter paper is made of cellulose.

In the case of tissues containing hard substances (bones, teeth), treatment with decalcifying solution containing formic acid is not recommended.

Notes

- Wear protective gloves and, if necessary, other protective clothing!
- Inspect blister packaging in advance and do not use embedding cassettes from damaged or previously accidentally opened blister packaging.
- Do not use damaged or already used embedding cassettes.
- Use embedding cassettes only within the specified shelf life.
- The further storage or disposal of the used or contaminated embedding cassettes is carried out according to the regulations of the facility in the suitable disposal container.

Application

Prepare the opened embedding cassette for sample collection and ensure that the filter paper is correctly inserted. Place the removed sample in the cassette and close the lid.

Storage

Store dry at +5°C to 25°C and protect from direct sunlight and humidity.

All serious incidents related to the product shall be reported to the manufacturer and the responsible authority of the member state where the user and/or patient is registered.

Symbol meaning

	Item no.		Sterilization method
	Chargenbezeichnung in YYX0000		Protect from sunlight
	Shelf life in YYYY-MM		Protect from rain
	Manufacturer		Do not use if packaging is damaged
	For single use only		Follow information documents
	Do not sterilize again		Storage temperature
	Sterile barrier system with protective packing outside		IVD product

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KABE LABORTECHNIK GmbH