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IFU\_GK\_0008\_RevI2026\_ML

## Instructions for use for Capillary blood collection GK

apply to the products Capillary blood collection systems GK unsterile in different preparations and sizes

### Intended use

The capillary blood collection GK is intended for the collection of a capillary blood sample from a suitable puncture site, e.g. the fingertip, the earlobe or, for paediatric use, from the lateral area of the heel or the scalp of the child.

The capillary blood collection GK is manufactured in different versions with regard to the amount of capillary blood to be collected as well as the preparation of the sample vessel and the capillary. For more detailed information regarding the preparation, please refer to the label of the respective products.

In principle, the capillary blood collection GK is intended for patients of all age groups and genders, provided that puncture with a suitable lancing device is possible. It is suitable for the collection, transport and processing of capillary blood in the clinical laboratory. Depending on the preparation of the capillary blood collection GK, different measured values in the blood can be determined and essential information about the patient's state of health can be provided.

All products from the product group Capillary Blood Collection GK are intended exclusively for single use in a professional environment by medically qualified personnel.

### Features

The capillary blood collection GK is a CE-marked IVD product, which is grouped in class A. It consists of a sample vessel, a capillary holder stopper, a sealing stopper with loop and a capillary. The stopper colours refer to the national or international colour code corresponding to annex F, ISO 6710:2017 depending on the preparation. The sample vessels are available in different designs. The capillaries are available in different lengths and inner diameters depending on the volume.

### Notes

- Wear protective gloves and, if necessary, other protective clothing!
- Use the product only within the indicated shelf life. The shelf life varies depending on the preparation.
- Any violent handling of the product must be avoided.
- Only use the product if complete filling of the capillary can be detected visually.
- The product is intended for capillary blood collection only.
- The product is for single use only!

### Application

Ensure that all necessary components of a capillary blood collection are prepared.

Prepare puncture site and do not touch after disinfection.

- 6) Perform puncture with a lancing device/lancet (products for performing the puncture are not part of the capillary blood collection GK).
- 7) Hold the capillary with vessel horizontally against the emerging blood drop until it is completely filled with blood.
- 8) Hold the sample vessel vertically after collection so that the blood can run out of the capillary into the vessel.
- 9) By gently tapping the tube on a solid surface, the blood remaining in the capillary can be completely emptied into the vessel.
- 10) Finally, remove the capillary with the capillary stopper and dispose it according to in-house regulations. Close the tube with the attached stopper and mix it carefully without foam formation.

### Storage

Store dry at +5°C to 25°C and protect from direct sunlight and humidity.

**All serious incidents related to the product shall be reported to the manufacturer and the responsible authority of the member state where the user and/or patient is registered.**

### Symbol meaning

|                                                                                     |                       |                                                                                     |                              |
|-------------------------------------------------------------------------------------|-----------------------|-------------------------------------------------------------------------------------|------------------------------|
|  | Item number           |  | Protect from sunlight        |
|  | Lot in YYX0000        |  | Protect from rain            |
|  | Shelf life in YYYY-MM |  | IVD product                  |
|  | CE-mark               |  | Storage temperature          |
|  | Manufacturer          |  | Follow information documents |
|  | For single use only   |                                                                                     |                              |

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KABE LABORTECHNIK GmbH