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Instructions for use Untreated Specimen container

apply to the products untreated specimen containers in various designs

Product description/Intended use

The products untreated specimen containers are used to hold solutions, substances or specimen material. The products are primary container for storage and transport purposes, which can be further processed in the laboratory. When using the specimen containers for sample material from the human body, important information about the patient's state of health can be obtained.

All products from the product group neutral sample containers are intended exclusively for single use in a professional environment by qualified laboratory personnel.

Features

The untreated specimen container is a CE-marked IVD product, which is grouped in class A. The products are available in different sizes and shapes and with different stopper colours and types.

Notes

- When handling hazardous substances or diagnostic samples, wear protective gloves and, if necessary, other protective clothing!
- Do not use damaged or already used specimen containers.
- Any violent handling of the product must be avoided.
- Use untreated specimen container only within the specified shelf life.
- The further storage or disposal of the used or contaminated specimen container with saline solution is carried out according to the regulations of the facility in the suitable disposal container.
- The product is for single use only!!

Application

Open the lid of the container, if it is not separate, and add the desired material using the appropriate tools. Close the container. Further steps depend on the type of use and are not part of these instructions for use.

Storage

Store dry at +5°C to 25°C and protect from direct sunlight and humidity.

All serious incidents related to the product shall be reported to the manufacturer and the responsible authority of the member state where the user and/or patient is registered.

Symbol meaning

	Item no.		Protect from sunlight
	Lot in YYX0000		Protect from rain
	Shelf life in YYYY-MM		IVD product
	CE-mark		Storage temperature
	Manufacturer		Follow information documents
	For single use only		

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KABE LABORTECHNIK GmbH