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Instructions for use Prepared specimen containers and reaction vessels

apply to the products prepared specimen vessels with in various preparations and sizes

Product description/Intended use

The products prepared specimen containers and reaction vessels are used to collect blood from a previously obtained blood sample. The prepared specimen containers and reaction vessels are available in various designs with respect to volume, shape and the preparation of the sample vessel.

In principle, the prepared specimen container and reaction vessel is intended for patients of all age groups and genders. It is suitable for the collection, transport and processing of blood in the medical laboratory. Depending on the preparation, different measured values can be determined in the blood and essential information about the patient's state of health can be obtained. All products from the product group prepared specimen container and reaction vessels are intended exclusively for single use in a professional environment by medically qualified personnel.

Features

Prepared specimen container and reaction vessels are CE-marked IVD products, which are grouped in class A. They each consist of a specimen container/reaction vessel with a stopper. The stopper colours correspond to the national or international colour code according to Annex F, ISO 6710:2017, depending on the preparation. The specimen container and reaction vessels are available in various designs as well as filling volumes.

Notes

- Wear protective gloves and, if necessary, other protective clothing!
- Do not use damaged or already used prepared specimen containers and reaction vessels.
- Any violent handling of the product must be avoided.
- Use prepared specimen containers and reaction vessels only within the specified shelf life. The shelf life varies depending on the preparation.
- The further storage or disposal of the used or contaminated specimen containers and reaction vessels is carried out according to the regulations of the facility in the suitable disposal container.
- The product is for single use only!

Application

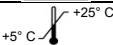
- Steps of the previous blood collection are not part of these instructions for use.
- Hold the specimen container/reaction vessel as vertically as possible during filling or place it in a suitable rack to prevent the sample from leaking. The respective filling volume must be observed.
- Further specimen processing steps are not part of these instructions for use.

Storage

Store dry at +5°C to 25°C and protect from direct sunlight and humidity.

All serious incidents related to the product shall be reported to the manufacturer and the responsible authority of the member state where the user and/or patient is registered.

Symbol meaning

	Item no.		Protect from sunlight
	Lot in YYX0000		Protect from rain
	Shelf life in YYYY-MM		IVD product
	CE-mark		Storage temperature
	Manufacturer		Follow information documents
	For single use only		Unsterile product

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KABE LABORTECHNIK GmbH