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Instructions for use for Cannula for blood collection (security)

apply to the products Sangocan® (security), Combi-Sangocan® (security) and AM G 46 K (security)

Intended use

The product Cannula for blood collection (security) is a sterile, single-use product intended for the collection of a venous blood sample from the human body. The blood sample is taken through a short-term contact of the stainless steel cannula externally with the blood vascular system. A suitable sample vessel is connected to the cannula, which is used to collect, transport and, if necessary, store the blood sample that has been taken.

The application is intended to be used in a professional environment by medically trained personnel.

The use of the cannula for blood collection (security) is generally intended for patients of all ages and genders, insofar as contact through the sterile cannula needle with a vein is possible. The venous blood sample taken with the medical device can provide a wide range of information needed for the diagnosis, prevention, monitoring, prediction, prognosis, treatment or relief of a disease.

Features

The Cannula for blood collection (security) is a CE-marked, EO-sterilized, pyrogen-free individually packaged medical device for single-use. The stainless steel cannulas with a triple facet cut in sizes 20G (0.9 mm), 21G (0.8 mm) and 22G (0.7 mm) have a length of 1 and 1 ½ inch (25 mm and 38 mm).

The needles are each connected to an adapter for the suitable blood collection system: needle-holder KABEVETTE® G, needle-holder Sangocan® and combination-holder Sangocan®.

The variants are available with or without safety mechanism that securely encloses the cannula after use.

Notes

- Wear protective gloves and if necessary, other protective clothing!
- Inspect blister pack in advance and do not use cannulas from damaged or previously accidentally opened blister packs. These must be disposed of immediately.
- Do not use bent, damaged or already used cannulas.
- Use cannulas only within the specified shelf life.
- If injury occurs due to damaged packaging or improper use, seek medical advice as there is a risk of infection and follow other safety measures of your facility if necessary.
- Any violent handling of the product must be avoided.
- Do not make any unnecessary sudden movements after removing the protective cap. **Do not reattach the protective cap at any time.**
- The product is only intended for the associated blood collection system.
- After collection, dispose the cannula (with activated security mechanism) as vertically as possible with the needle tip held upwards to avoid blood dripping.
- Dispose of the used or contaminated cannula and accessories in the appropriate sharps disposal container.
- The product is for single use only! Multiple use of the product can lead to injuries, infections or death. The product is not to be sterilized again.

Application

Ensure that all necessary components of a venous blood collection are prepared.

Apply tourniquet, prepare puncture site and do not touch after disinfection.

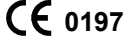
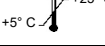
- 1) Pull the blister packaging apart at the area marked with an arrow using your thumbs and forefingers and remove the cannula.
- 2) Insert the first KABEVETTE® G or Primavette® S/V sampling tube into the cannula adapter with a slight clockwise rotation and light pressure.
- 3) Remove the protective cap of the cannula and puncture the vein with the ground edge of the cannula pointing upwards. During blood collection, hold the adapter in one hand and gently pull the piston rod with the other hand until the piston is fixed in the grid.
- 4) Remove the tourniquet. Wait a few seconds until the residual vacuum is compensated.
- 5) Changing: Remove KABEVETTE® G or Primavette® S/V from the adapter by turning it slightly to the left and insert a new tube. Then proceed according to the principle described above.
Further information on the aspiration technique or pre-aspiration of the tube for the vacuum technique can be found in the instructions for use for the KABEVETTE® G or Primavette® S/V.
- 6) After the last tube is filled, rotate the tube out of the adapter as before and pull the cannula out of the vein.
- 7) Now, the needle shield can be activated with a simple one-handed movement, e.g. with the aid of a stable surface. The needle guard is pressed over the cannula until it audibly engages in the protective shield. The cannula is now irreversible enclosed in the needle shield.
- 8) Dispose of secured unit in a sharps disposal container in accordance with operating instructions.

Storage

Store dry at +5°C to 25°C and protect from direct sunlight and humidity.

All serious incidents related to the product shall be reported to the manufacturer and the responsible authority of the member state where the user and/or patient is registered.

Symbol meaning

	Item number		Sterilization method Ethylene oxide
	Lot in YYX0000		Protect from sunlight
	Shelf life in YYYY-MM		Protect from rain
	CE-mark + notified body		pyrogen-free
	Manufacturer		Do not use if packaging is damaged
	For single use only		Follow information documents
	Do not sterilize again		Storage temperature
	Single sterile barrier system with protective outer packaging		Medical device
	Simple sterile barrier system		Unique product identification

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KABE LABORTECHNIK GmbH